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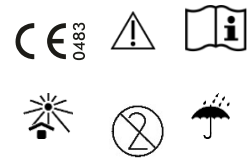
Shaver blades  
steril

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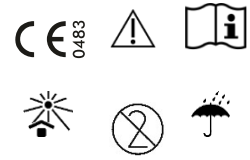
Copyright by REMA Medizintechnik GmbH  
All rights reserved  
Technical changes reserved

Rev. V004

Date: 23.08.2022

## ENGLISH

### Caution

Please read all information contained in this insert. Incorrect handling and care as well as misuse can lead to premature wear or risks for the patient and user.

## 1 Scope

These operating instructions are applicable for REMA Shaver Blades sterile (product group 2AG4) risc class IIa:

REF 15-6XXXXS – 15-79XXXX.

## 2 Material

Tube: Stainless Steel 1.4301  
Blade: Stainless Steel 1.4021  
Top Part: Shaver Blades for  
LINVATEC/Arthrex and Arthrex APS II - PMP  
Dyonics EP1/Stryker SE-3/SE-4/SE-5 - Aluminium  
Dyonics PS 3500 EP/Stryker Formula - PPSU  
Aesculap/Arthronet - 1.4305

## 3 Intended use

The REMA Shaver Blades sterile are used for removal (cutting and milling) of delicate soft tissue or cartilage of small and large joints in arthroscopic surgery.

## 4 Indication

The following general indications belongs for the use of the arthroscopic treatment of the Shaver system:

- resection of meniscus tissue
- synovia resection
- resection of scar tissue
- aspiration of loose fragments and small meniscus particle
- cartilage debridement, removal of large cartilage fringes
- milling of exposed bones for induction of replacement cartilages
- notch plastic
- removal of osteophytes
- smoothing of drill holes entries

## 5 Contraindication

- local or generalized infections
- immune suppressive therapy
- coagulation disorders, embolism or thrombosis
- serious systemic disease that make a survival after the surgery unlikely
- if an arthroscopic treatment is not indicated in general

Product-specific contraindications are not existing at the moment.

## 6 Normal use

Only skilled medical personnel and personnel trained in this product are permitted to use Shaver Blades.

### Please note before use:

Sterile Shaver Blades are designated for single use. Sterile Shaver Blades must be discarded after use. Please note the information on the product labeling.

A visual test must be performed before each use (see section "visual inspection"). It must be ensured that Instrument-Insert is inserted firmly in the handle.

The sterile Shaver Blades must not be used past the expiration date! See expiration date at the label!

IFU-16-V004-GER-ENG-ESP

### Application:

The type of appropriate attachments is selected depending on the indication and scope.

Blades bound for soft tissue should not be applied on the bone and vice versa.

Rema Shaver Blades have to be connected to shaver handpieces. They are used for the following shaver systems:

- REMA/ Linvatec/ Arthrex
- Arthrex APS II/ REMA
- Atlantec/ Dyonics EP1
- Aesculap
- Stryker Formula

Please check before use if shaver blade is suitable for your system.

During use the shaver blades must be completely surrounded with irrigation fluid. Always ensure an adequate flow of irrigation fluid (at least 100 ml/min) to the joint.

### Optimal Speed:

Design:	rpm:
Blades	800 – 2000
Burrs	2500 – 3500

### Assembly and Operation:

- Position the shaver blade with the pen in the leadership of the handpiece.
- Shaver Blades are controlled by the foot pedal
- Please follow for introducing the REMA Shaver Blades in the Dyonics EP1 - handpiece the following instructions:

### Assembly of Dyonics EP1 Shaver Blades

For insertion into the handpiece

1. Turn the knurled clockwise until it stops.
2. Blades in handpiece insert (pin into the small hole)
3. Turn the knurled counter-clockwise until stop to remove them from the handpiece
4. Turn the knurled right
5. Remove the blade from handpiece

### ATTENTION:

- In case of **curved shaver blades** there must be determined the **position by a turning motion with a slight axial pressure** in which it is possible to insert or dismantle the blade effortless from the tube.

## 7 Safety notice

### WARNINGS AND PRECAUTIONS!

- **Speed:** For the proper functioning of shaver blades meet the specified ranges of rotation. At too high speed, there is a risk that the removed material clogging the shaver. The speeds specified must be adhered to for safety and optimal use for. If anything is unclear, contact the manufacturer.

- **Damage:** If shaver blades operated in the dry state, this can result a damage (for example the blade).

The blades may in the rotating state don't come into contact with metal (eg, surgical instruments, arthroscope, etc.), otherwise it will cause damage.

The dismantling and the following assembling of the shaver blade (blade into tube) and the inserting of the shaver blade into the hand piece must be done carefully, in order to avoid damage to the shaver blades and / or injuries to the patient or surgical personnel. Shaver Blades may be damaged if excessive force is applied or by contact with metal!

Before you use the shaver blades they must be checked for damage. The Shaver Blade must be replaced by a new one (even during operation) if damage of any kind exists.

- **Cutting performance:** If a sufficient cutting power is not given, the Shaver Blades may no longer be used and must be replaced. See section "visual inspection".

### - Sterile Packaging

Take attention to the integrity of the outer and inner packaging by sterile Shaver Blades. The sterility is only guaranteed as long as the packaging is undamaged and not opened.

## 8 Storage / transportation

Sterile Shaver blades must be stored in a dry location, protected from dust, moisture and sunlight at a moderate temperature in the range of 7°C to 32°C, see temperature range at the label. They should be individually stored in a protective container with individual compartments. The instruments must always be handled with the utmost care during transportation and storage. This applies in particular for fine tips and other sensitive areas.

## 9 Visual inspection

Before each use, the outer and inner shaft should be checked for pressure points or damage. Any damaged blades must not be used and replaced by new ones. Especially the cutting edges should be inspected. They must be sharp and in good order. See section "safety notice – cutting performance".

## 10 Returns

Returns will be accepted only if they are marked as "hygienically safe" or "not decontaminated" and have been securely packaged for shipping.

**Attention:** Returned products must pass the complete re-processing process before being returned for repair.

## 11 Liability / Warranty Claim / Repair and modification

REMA Medizintechnik GmbH as manufacturer and seller of these products assumes no liability for immediate or ensuing damages resulting from improper use or handling, especially due to failure to use as directed or due to combination with products of other manufacturers.

Never attempt to perform repairs yourself. Service and repair work may only be performed by persons qualified and trained accordingly. Defective Shaver Blades may be sent to REMA Medizintechnik GmbH for repair. Unauthorized modifications and repairs are strictly prohibited and will entail invalidation of the manufacturer's warranty and liability.

## 12 Disposal

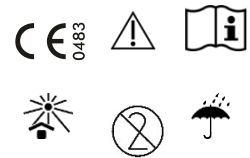
Shaver blades and the packaging material must be disposed of in accordance with the regulations and laws specific to the country in which they are used.

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Copyright by REMA Medizintechnik GmbH  
All rights reserved  
Technical changes reserved

Rev. V004

Date: 23.08.2022

## ENGLISH

### 13 About these Instructions of Use

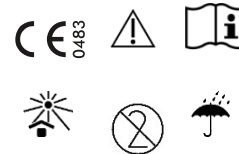
Throughout the period of use of medical devices, the Instructions for Use must be kept freely accessible for every user.

Explanation of the symbols:

	Attention. Obey the documents accompanying the product regarding important safety relevant information as warnings and precautions.
	Medical Device
	Follow instructions for use
	Article number
	Batch code
	CE-Mark of the notified body Mdc medical device certification GmbH, Stuttgart, Germany
	Sterilised by irradiation
	Not reusable
	If the packaging is damaged the product must not be used.
	U S Federal law restricts this device to sale by or on the order of a medical practitioner
	expiration date
	Protect from moisture
	Keep away from sunlight
	Do not dispose of residual waste
	Manufacturer

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Derechos de autor de REMA Medizintechnik GmbH
Marcado CE del organismo notificado Mdc medical device certification GmbH, Stuttgart, Germany