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**INSTRUCTIONS FOR USE**
**Retractors, self-retaining**

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Tel. +49(0)7461-92393-0  
Fax. +49(0)7461-92393-33  
www.dewimed.de

**IMPORTANT INFORMATION PLEASE READ BEFORE USE**
**Products**

18.xxx.xx; 40.xxx.xx; 54.xxx.xxx; 56.xxx.xx; 66.xxx.xxx; 68.xxx.xxx

**Intended use**

Retractors with fixation belong to the group of holding and holding away instruments. Their task is to hold tissues, organs and bones or spread wound edges.

**! WARNINGS**

- All retractors are sold non-sterile.
- All retractors must be cleaned, disinfected and sterilized before and after use.
- Do not use highly alkaline or acidic cleaning solutions

**Restrictions on reprocessing**

- Regular reprocessing (with ultrasound, manual cleaning, sterilization) has minimal impact on the instruments
- Inspect the products carefully before each use to ensure their proper functioning.
- The lifespan of the products is usually determined by wear and tear or damage during surgical use.
- Signs of damage and wear and tear on a product may include corrosion (rust, pitting), discoloration, severe scratches, peeling, wear and tear. The improper functioning of the products, products with unrecognizable markings, missing or removed (abraded) article numbers as well as damaged and heavily worn products must not be used.
- Do not use damaged instruments. They must be disposed of in accordance with the usual hospital requirements.

**Guide**
**Place of use**

- Remove dirt with a disposable cloth/paper towel.
- All instruments should be rinsed with a high-pressure water jet.

**Transport**

- It is recommended to transport contaminated instruments in a closed clean container.
- It is recommended to prepare the product no later than 2 hours after use.

**Preparation for decontamination**

Wear personal protective equipment (gloves, a water-repellent gown, face mask or goggles; and also follow the instructions of the manufacturer of the cleaning agent or disinfectant).

**Pre-cleaning**

**Equipment:** Personal protective equipment (as listed above), cold tap water (20 ± 2 °C) at least in drinking quality (according to the RKI guideline), high-pressure water gun.

Rinse the products with a high-pressure water gun for at least 1 minute, paying attention to critical and hard-to-reach areas.

**Cleaning: Manual**

- Before manual cleaning and subsequent manual disinfection: Perform a pre-cleaning (see section Pre-cleaning).
- **Equipment:** Cleaning solution: multi-stage enzymatic cleaning agent (e.g. Sekusept® MultiEnzyme from ECOLAB (www.ecolab.com)), plastic brushes (e.g. Interlock #09095, #09084, #09050, #09068) container for cleaning solution, tap water (20 ± 2°C) at least in drinking quality (according to the RKI guideline). Dip the product in the cleaning solution.
- 1. The cleaning solution must be prepared according to the manufacturer's instructions (e.g. Sekusept® MultiEnzyme 2% solution).
- 2. Dip the product in the cleaning solution.
- 3. The soaking time must comply with the manufacturer's instructions (at least 15 minutes for Sekusept® MultiEnzyme 2% solution).
- 4. During the soaking period, the product remains immersed in the solution; at the same time, all joints, cavities and all hard-to-reach areas are cleaned with a soft brush to remove visible dirt.
- 5. **Pay attention to** critical and hard-to-reach areas where the effectiveness of cleaning cannot be visually verified.
- 6. Fill a 20 ml syringe with cleaning solution and rinse the joints, cavities and all hard-to-reach areas and all accessible areas of the instrument. Repeat this step 5 times.
- 7. **Pay attention to** critical and hard-to-reach areas where the effectiveness of cleaning cannot be visually verified.
- 8. Remove the instruments from the cleaning solution.
- 9. Rinse the product with running tap water for 1 minute, rinsing all surfaces for at least 10 seconds.
- 10. Check the cleanlines; in case of visible dirt residues, repeat the above steps.

**Disinfection – Manual**

**Equipment:** Peroxide-based disinfectant (e.g. Sekusept® aktiv from ECOLAB (www.ecolab.com)), container for disinfectant solution, tap water (20 ± 2°C) at least in drinking quality (according to the RKI guideline), ultrasonic bath (35 kHz), deionized water (free of optimal microorganisms according to the KRINKO/BfArM recommendation), lint-free wipes or compressed air gun with compressed air for medical purposes (according to the European Pharmacopoeia), a sterile 20 ml syringe (e.g. Braun # 4606205V)

1. The disinfectant solution (e.g. Sekusept® active 2% solution) must be prepared in an ultrasonic bath according to the manufacturer's instructions.
2. Immerse the product completely in the disinfectant solution.
3. The soaking time must comply with the manufacturer's instructions (at least 10 minutes for Sekusept® active 2% solution).
4. Fill a 20 ml syringe with disinfectant and rinse all joints, cavities and other hard-to-reach areas of the product at least 5 times during the soaking period.
5. **Pay attention to** critical and hard-to-reach areas where the effectiveness of cleaning cannot be verified by visual inspection.
6. After a 5-minute soaking time, start a 5-minute ultrasonic cleaning.
7. Turn off ultrasonic cleaning.
8. Remove the product from the disinfectant solution.
9. Rinse the product with running tap water for at least 1 minute.
10. Repeat the rinsing process with deionized water.
11. Wipe the product with a lint-free, single-use cloth or manual, or dry it with compressed air.

**Cleaning and disinfection: Automated**

- If there are visible dirt residues in crevices, openings, threads and other hard-to-reach areas: Perform a pre-cleaning (see section Pre-cleaning) and proceed with a manual cleaning.
- **Equipment:** System for thermal disinfection (washing machine / disinfection system) according to DIN EN ISO 15883-1/-2 with thermal program (temperature 90-93 °C), mild alkaline cleaning agent (e.g. Sekumatic® MultiClean 200 ml), lint-free wipes or compressed air gun with compressed air for medical purposes (according to the European Pharmacopoeia).

1. Place all parts of the product in a suitable cleaning sieve/trolley for thermal disinfection (washing machine / disinfection system) or connect the surgical instruments to Lumina with rinsing inserts of a suitable cleaning sieve/trolley.
2. Close the device and start the thermal program; the program parameters are shown in the table below.
3. Remove the product at the end of the program.
4. Check that the product is completely dry. If necessary, wipe the product with a lint-free single-use cloth or manual, or dry it with compressed air.

Program-step	Water	Dosage	Time	Temperature
Pre-rinse	cold		5 Min.	
Cleaning agent dosing		According to the manufacturer's instructions		According to the manufacturer's instructions
Cleaning	Deionized water		10 Min.	55°C

Program-step	Water	Dosage	Time	Temperature
Rinse	Deionized water		2 Min.	
Disinfection	Deionized water		Ao -Value >3000 <sup>4</sup> °e.g. 5 Min, 90 °C	
Dry			15 Min.	Until 120°C

<sup>1</sup> Under certain circumstances, authorities may require compliance with other procedures (parameters for disinfection) for their area of responsibility

#### Maintenance, Inspection and testing

- Equipment:** Silicone-free instrument oil suitable for steam sterilization for use in medical devices (e.g. Sterilit® spray from B. BRAUN, Aesculap or other suppliers).
- Inspect all components of the product under good lighting conditions for possible dirt residues. For components with visible dirt residues, repeat manual cleaning (including pre-cleaning) and automated cleaning and disinfection or manual disinfection.
  - Check the product for completeness, damage and functionality.
  - Check elongated products for twisting.
  - Do not use damaged instruments.
  - Lubricate the moving parts of the instrument with a silicone-free instrument oil suitable for steam sterilization.

#### Packaging

Packaging in a sterile barrier system according to DIN EN ISO 11607 or DIN 58953 (double-wrapped sterilization paper was used for validation)

#### Sterilization

**Equipment:** Sterilization device according to DIN EN 13060 and/or DIN EN 285.

- Baking vacuum, 134 °C and a sterilization time of at least 3 minutes (longer holding times and temperatures up to 141 °C are possible).
- Sterilization of the product in a disassembled state, if possible
  - Place the packaged product in the sterilization chamber.
  - Launch the program.
  - After finishing the program, remove the product and let it cool.
  - Check that the packaging is tightly closed and dry.

#### Storage

Storage duration and shelf life are determined by the user.

- Instruments must be stored in a dry, clean and moisture-free environment away from direct sunlight.
- Do not store in places where chemicals release corrosive fumes or where changing temperatures or humidity levels cause condensation to form on the instruments.

#### Additional Information

Only validated processes may be used for the reprocessing of medical devices

#### Symbols used

	Attention		Manufacturer		Manufacturing date
	Medical device		Non-sterile		Quantity in packaging
	Article number		Lot-number		Follow instruction for use
	Keep dry		Keep away from sunlight		
	CE-Mark of the Notified Body mdc medical device certification GmbH, Stuttgart, Germany		Prescriptive medical device (USA)		

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