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USER INSTRUCTIONS
REMA Shaver Blades Steril

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Tel. +49(0)7461-92393-0
 Fax. +49(0)7461-92393-33
 www.dewimed.de

IMPORTANT INFORMATION PLEASE READ BEFORE USE

Caution Please read all information contained in this insert. Incorrect handling and care as well as misuse can lead to premature wear or risks for the patient and user.

Scope

These operating instructions are applicable for REMA Shaver Blades sterile:

REF 15-6XXXX – 15-79XXX.

Intended use

The REMA Shaver Blades sterile are used for removal (cutting and milling) of delicate soft tissue or cartilage of small and large joints in arthroscopic surgery.

Indication

The following general indications belongs for the use of the arthroscopic treatment of the Shaver system:

- resection of meniscus tissue
- synovia resection
- resection of scar tissur
- aspiration of loose fragments and small meniscus particle
- cartilage debridement, removal of large cartilage fringes
- milling of exposed bones for induction of replacement cartilages
- notch plastic
- removal of osteophytes
- smoothing of drill holes entries

Contraindication and Residual risks

- Local or generalized infections
- Immune suppressive therapy
- Coagulation disorders, embolism or thrombosis
- Serious systemic disease that make a survival after the surgery unlikely
- If an arthroscopic treatment is not indicated in general. Product-specific contraindications are not existing at the moment.

Normal Use

Only skilled medical personnel and personnel trained in this product are permitted to use REMA Shaver Blades.

Please note before use:

Sterile Shaver Blades are designated for single use. Sterile Shaver Blades must be discarded after use. Please note the information on the product labeling.

A visual test must be performed before each use (see section "visual inspection"). It must be ensured that Instrument-Insert is inserted firmly in the handle.

The handpiece used must be checked in accordance with the detailed instructions in the corresponding instructions for use. In the event of damaged handpiece, a new handpiece should be used to avoid damage to the REMA Shaver Blades.

The sterile Shaver Blades must not be used past the expiration date! See expiration date at the label!

Application:

The type of appropriate attachments is selected depending on the indication and scope.

Blades bound for soft tissue should not be applied on the bone and vice versa.

REMA Shaver Blades have to be connected to shaver handpieces. They are used for the following shaver systems:

- REMA / Linvatec / Arthrex
- Arthrex APS II / REMA
- Atlantec / Dyonics EP1
- Aesculap
- Stryker Formula

Please check before use if shaver blade is suitable for your system.

During use the REMA Shaver Blades must be completely surrounded with irrigation fluid. Always ensure an adequate flow of irrigation fluid (at least 100 ml/min) to the joint.

Speed range:

Design:	Rpm
Blades	800 – 2000
Burrs	2500 – 3500

Assembly and Operation:

- Position the shaver blade with the pen in the leadership of the handpiece.
- REMA Shaver Blades are controlled by the foot pedal
- Please follow for introducing the REMA Shaver Blades in the Dyonics EP1 - handpiece the following instructions:

Assembly of Dyonics EP1 REMA Shaver Blades

For insertion into the handpiece

1. Turn the knurled clockwise until it stops.
2. Blades in handpiece insert (pin into the small hole)
3. Turn knurl to the left as far as it will go

To remove from the handpiece

1. Turn the knurled right
2. Remove the blade from handpiece

ATTENTION In case of **curved REMA Shaver Blades** there must be determined the position by a **turning motion with a slight axial pressure** in which it is possible to insert or dismantle the blade effortless from the tube.

Safety notice
! WARNING AND PRECAUTIONS!

- **Speed:** For the proper functioning of REMA Shaver Blades meet the specified ranges of rotation. At too high speed, there is a risk that the removed material clogging the shaver. The speeds specified must be adhered to for safety and optimal use for. If anything is unclear, contact the manufacturer.
- **Damage:** If REMA Shaver Blades operated in the dry state, this can result a damage (for example the blade).

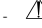
The blades may in the rotating state don't come into contact with metal (eg, surgical instruments, arthroscope, etc.), otherwise it will cause damage.

The dismantling and the following assembling of the shaver blade (blade into tube) and the inserting of the shaver blade into the hand piece must be done carefully, in order to avoid damage to the REMA Shaver Blades and / or injuries to the patient or surgical personnel. REMA Shaver Blades may be damaged if excessive force is applied or by contact with metal!

Before you use the REMA Shaver Blades they must be checked for damage.

The Shaver Blade must be replaced by a new one (even during operation) if damaged of any kind exists.

Cutting performance: If a sufficient cutting power is not given, the REMA Shaver Blades may no longer be used and must be replaced. See section "visual inspection"

Sterile Packaging:
 Take attention to the integrity of the outer and inner packaging by sterile Shaver Blades. The sterility is only guaranteed as long as the packaging is undamaged and not opened.

Storage / packaging / transportation / handling

 Sterile REMA Shaver blades must be stored in a dry location, protected from dust, moisture and sunlight at a moderate temperature. They should be individually stored in a protective container with individual compartments. The instruments must always be handled with the utmost care during transportation and storage. This applies in particular for fine tips and other sensitive areas.

Visual Inspection

Before each use, the outer and inner shaft should be checked for pressure points or damage. Any damaged blades must not be used and replaced by new ones. Especially the cutting edges should be inspected. They must be sharp and in good order. See section "safety notice – cutting performance".

Returns

Returns will be accepted only if they are marked as "hygienically safe" or "not decontaminated" and have been securely packaged for shipping.

Attention: Returned products must pass the complete reprocessing process before being returned for repair.

Liability / Warranty Claim / Repair and modification

DEWIMED Medizintechnik GmbH as manufacturer and seller of these products assumes no liability for immediate or ensuing damages resulting from improper use or handling, especially due to failure to use as directed or improper processing or due to combination with products of other manufacturers.

Never attempt to perform repairs yourself. Service and repair work may only be performed by persons qualified and trained accordingly. Defective REMA Shaver Blades may be sent to DEWIMED Medizintechnik GmbH for repair. Unauthorized modifications and repairs are strictly prohibited and will entail invalidation of the manufacturer's warranty and liability.

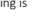
Disposal

REMA Shaver Blades and the packaging material must be disposed of in accordance with the regulations and laws specific to the country in which they are used.

About this Instructions of use

Throughout the period of use of medical devices, the Instructions for Use must be kept freely accessible for every user.

Explanation of symbols:

	Article number		Medical device		Quantity in packaging
	Lot-number		Manufacturer		Manufacturing date
	Follow instruction for use		Not for reuse		Keep dry
	Keep away from sunlight		EO sterilized		Attention
	Usable until		Do not sterilize again		Double sterile barrier system
	Do not use if packaging is damaged		Do not dispose of in residual waste		
	Prescriptive medical device (USA)		CE-Mark of the Notified Body mdc medical device certification GmbH, Stuttgart, Germany		

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