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IMPORTANT INFORMATION PLEASE READ BEFORE USE

Please read all information contained in this insert. Incorrect handling and care as well as misuse can lead to premature wear or risks for the patient and user.

Scope

These instructions for use are applicable for the Shaver System A9000 handpiece:

SH-T001.1.2

Purpose

The A9000 handpiece is controlled by the A9000 control unit and rotates a shaver blade that is used to remove (cut and ream) fine soft tissue or cartilage from small and large joints during arthroscopy.

Indication

General indications for use of the Shaver System A9000 for arthroscopic treatment include:

- Resection of meniscal tissue
- Resection of synovial tissue
- Resection of scar tissue
- Suction of free joint bodies and small meniscus particles
- Cartilage smoothing, removal of larger cartilage fringes
- Milling of exposed bone for the induction of replacement cartilage
- Notchplasty
- Removal of osteophytes
- Smoothing of drill channel inlets

Contraindications

The Shaver System A9000 is contraindicated in:

- Local or generalized infections
- Immunosuppressive therapy
- Blood clotting disorders / tendency to embolism or thrombosis
- Severe systemic diseases that make survival after surgery unlikely
- If arthroscopic treatment is generally not indicated

Intended use

The intended use of the A9000 Shaver System is for use during arthroscopic procedures.

The A9000 control unit may only be used to control the Shaver System. The A9000 control unit is synchronized with the A9000 handpiece, the A9000 Bluetooth® receiver and the A9000 foot control and is only used for this purpose. If this system is only connected to these elements and the power supply is ensured, it can be used as an arthroscopy device.

Combination products

The Handpiece A9000 is compatible with the following products from Dewimed Medizintechnik GmbH:

Article no.	Description	Function
SH-T001.1	Shaver Control Unit A9000	Control unit for regulating the motor function and power supply
SH-T001.1.1.3	Shaver Foot pedal A9000	Hands-free control of speed and direction of rotation
SH-T001.1.1.4	Shaver Connecting Cable A9000	Transmission of power and control signals between control unit and handpiece
SH-T001.1.1.5	Shaver Bluetooth Receiver A9000	Wireless connection between foot pedal and control unit
15-60XXX	Shaver Blades Coupling Variant 0; IFU-15 / IFU-16 See section "Insertion and locking the Shaver Blades"	
15-61XXX	Shaver Blades Coupling Variant 1; IFU-15 / IFU-16 See section "Insertion and locking the Shaver Blades"	

Function:

The shaver blades are driven by the Handpiece A9000 in a right/left rotation as well in a pendulum movement (oscillation; alternatively left, right).


The Handpiece A9000 is connected to a locked blade and the Control unit A9000 with the Connecting cable A9000 for power supply and electronic control. The electric motor of the Handpiece A9000 is controlled (direction and rotation speed) via the Footswitch A9000 connected to the Control unit A9000 (via Bluetooth® Receiver A9000), by the buttons on the handle of the Handpiece A9000 or the buttons on the Control unit A9000.

The removed shavings and tissue residues are suctioned through the inner channel of the shaver blade and the Handpiece A9000. Therefore, there is no need for work breaks to wash the joint, the suction unit shall be active and connected to the suction connection of the Handpiece A9000 with a hose.

Speed range:

	rpm
Right / left rotation	500 – 9000
Oscillation	500 – 4000

Description of the handpiece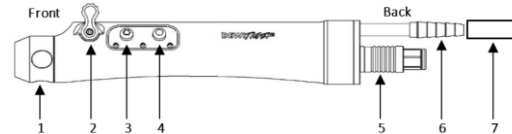

- 1: Chuck for the locking of Shaver Blades
- 2: Lever for stepless flow control
 - Lever back = increase suction volume
 - Lever to the front = reduce extraction volume
- 3: Button 1: dual function
 - If the Handpiece A9000 is NOT switched on:
Single press: start button
Press and hold: increase of rotational speed in 100 rpm jumps
 - If the Handpiece A9000 is switched on:
Single press: stop button
Press and hold: stop button
- 4: Button 2: dual function
 - If the Handpiece A9000 is NOT switched on:
Single press: button for working mode selection; clockwise rotation (right, front), counter-clockwise rotation (left, reverse) or oscillation
 - If the Handpiece A9000 is switched on:
Single press: stop button
Press and hold: stop button
- 5: Connection cable plug
- 6: Hose barb connection for suction
- 7: Suction hose

IMPORTANT! Function of the buttons

If the Handpiece A9000 is switched on and both buttons are pressed and held simultaneously it will switch off. If the Handpiece A9000 is switched on and both buttons are briefly pressed simultaneously it will switch off.

WARNING! Pressing buttons

If the Handpiece A9000 is on and either button is held down there is no increase or decrease in speed until the actuator is released and pressed again with the Handpiece A9000 off.

IMPORTANT! Commissioning

If the device is switched off with the functions of the buttons locked, the next time it is switched on, it will remain in the previous state.

IMPORTANT! Benachrichtigungston

Each time either button is pressed and held down, a "beep" is heard for every 100 rpm jump in the rotational speed setting.

WARNING! Speed auto tuning in oscillation mode

If the speed set at the time of selecting the oscillation mode is higher than 4000 rpm, it will automatically take the value of 4000 rpm during the work cycle, once you stop the Handpiece A9000 and switch to one-way rotation mode, it automatically returns to the previous value.


Commissioning

Connection of the Handpiece A9000


1. Connect the Handpiece A9000 to the Control unit A9000
2. Connect the suction hose to the Handpiece A9000
3. Adjust the suction cannal valve
4. Insert the shaver blade (burr) into the Handpiece A9000 and lock it in place

Connection of the Handpiece to the Control unit

1. Remove the immersion caps from both the Handpiece A9000 and the Connecting cable A9000
2. Identify the two terminals of the Connecting cable A9000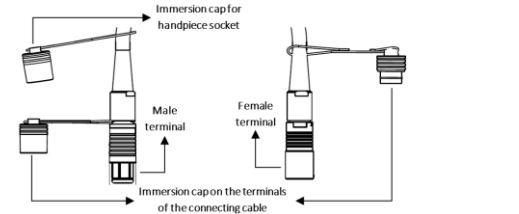

3. Align the red dot on the female terminal of the Connecting cable A9000 with the red dot on the connection socket for the Connecting cable A9000 of the Handpiece A9000
4. Carefully push the cable lug into the plug, until it is completely covered and a "click" sound is heard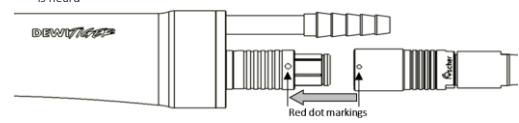

5. Align the point of the terminal of the Connecting cable A9000 with the red point of the connection plug for the Handpiece A9000 of the Control unit A9000

6. Carefully push the terminal of the Connecting cable A9000 into the connection plug until it is fully inserted and a "click" sound is heard

Connecting the suction hose to the Handpiece


1. Insert a suction hose into the Handpiece A9000 hose barb connection, the hose should cover as many barbs as its gauge will allow
2. Pull carefully on the suction tube to ensure that it is firmly connected to the Handpiece A9000

Important! Intended use

Failure to properly attach the suction hose to the spigot connection may result in improper suction due to air entering the channel.

Adjusting the suction duct

1. Slide the flow control lever towards the front of the Handpiece A9000 to close the suction channel
2. Slide the flow control lever towards the back of the Handpiece A9000 to open the suction channel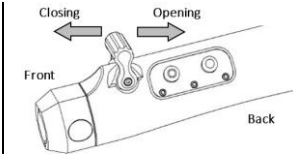


IMPORTANT! Intended use

- Keep the channel fully open for correct and continuous flow
- Adjust the flow volume with the pump in order to maintain a constant pressure in the working area
- Due to the particularities of each case, the quantitative determination of the power in the irrigation and suction system is determined by the user

Insertion and locking the Shaver Blades

The Handpiece A9000 is intended to work with Shaver Blade coupling variants 0 and 1 (REF 15-60XXX and 15-61XXX) manufactured by Dewimed Medizintechnik GmbH.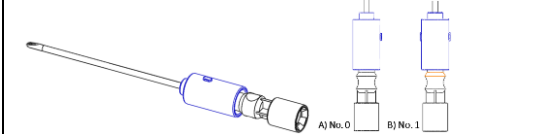

Variant A: Shaver Blade Coupling REMA, Linvatec; REF 15-60XXX


Variant B: Shaver Blade Coupling REMA, Arthrex; REF 15-61XXX

IMPORTANT! Shaver Blade Handling


Before inserting a Shaver Blade into the shaver system Handpiece A9000, it is important to follow proper handling procedures to ensure safety and maintain the integrity of the blade:

1. Inspect the blade: Before handling the shaver blade, visually inspect it for any signs of damage, such as bent or dull edges. Ensure that the blade is clean and free from any debris or contaminants
2. Use appropriate personal protective equipment (PPE): Wear gloves and any other required PPE, such as protective eyewear or a mask, to minimize the risk of injury and maintain hygiene
3. Handle with care: Shaver blades are sharp and delicate instruments. Handle them with care to avoid accidental cuts or damage. Hold the blade by the handle or designated non-cutting areas to minimize the risk of injury
4. Avoid direct contact with the cutting edge: Avoid touching the cutting edge of the blade with bare hands or any other objects to prevent injury and maintain the sterility of the blade


1. Align the guide grooves on the front of the Handpiece A9000 chuck HORIZONTALLY, rotating the chuck clockwise (right) as far as the device will allow and listen for a "click"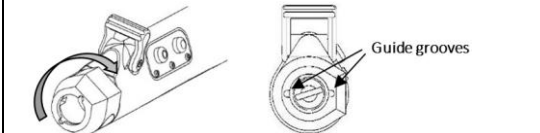


2. Insert the Shaver Blades into the collet of the A9000 handpiece using the guide base shown at the bottom inside the A9000 handpiece while aligning the Shaver Blades with the guide grooves on the front of the collet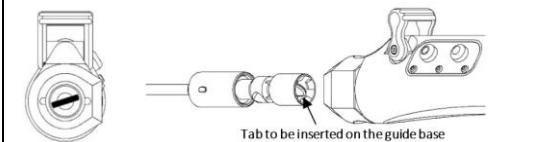

3. To lock the blade, rotate the chuck counter-clockwise (left) until its rear profile matches the profile of the Handpiece A9000 handle, the guide grooves should be aligned VERTICALLY, and a "click" should be heard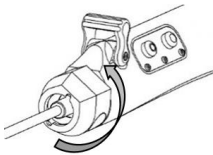

4. Check that the Shaver Blades are held firmly in position

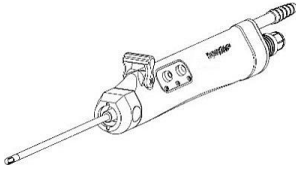WARNING!

Never attempt to mount a blade if:
 - The Handpiece A9000 is turned on
 - The Handpiece A9000 is connected to the Control unit A9000 and the Control unit A9000 is turned on

WARNING!

Risk of cut and/or infection

Never touch the sharp or jagged surface of the blade

IMPORTANT!

Intended use

Use the blade only in a liquid environment to avoid overheating or obstruction

Shutdown procedure

At the end of the use of the device during an arthroscopy procedure be sure to follow the following sequence of steps in order.

WARNING!

Risk of harm to the patient or user

1. Deactivate the Handpiece A9000 before removing the shaver blade from the inside of the patient
2. Carefully remove through the working channel

IMPORTANT!


The irrigation-aspiration system should be kept active throughout the procedure, unless otherwise instructed by the user, and after removal of the working instrument from the patient, the suction hose may be reconnected to other instruments or equipment being used

3. Place the Handpiece A9000 on a smooth and secure surface

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4. Press the turning on/off switch on the Control unit A9000 to turn off it
5. Remove the blade from the Handpiece A9000, following the disassembly procedure described in these instructions for use
6. Disconnect from the power supply
7. Disconnect the Connecting cable A9000 from the Control unit A9000 and Handpiece A9000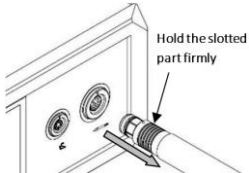

8. Attach the appropriate immersion caps on each connector to prevent substances from entering the connectors
9. Clean, disinfect and sterilize (if applicable) each component according to the instructions in this manual

Removing the shaver blade

1. Switch off the Control unit A9000


WARNING!


Risk of harm to the user

If the power supply to the Handpiece A9000 is not turned off, the user can activate it by mistake during the blade disassembly process.

2. To unlock the blade, align the guide grooves on the front of the Handpiece A9000 chuck HORIZONTALLY, rotating the chuck clockwise (right) as far as the device will allow and listen for a "click"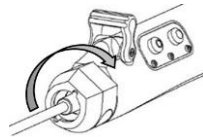

3. Carefully pull the blade out of the Handpiece A9000

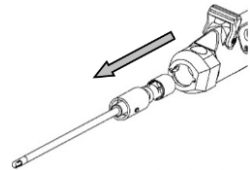Reprocessing – Cleaning, disinfection and sterilization

ATTENTION!

Completely open all suction lines for maximum exposure during reprocessing

IMPORTANT!

Expiry of the warranty

- If the appliance is reconditioned in any way other than as specified in these instructions, the warranty will be invalidated
- If the reconditioning specified in these instructions is not carried out or is only partially carried out, the warranty will be invalidated

IMPORTANT!

Limit for reprocessing

Appropriate reprocessing has a minimal effect on the Shaver System A9000. End of life is usually caused by wear and tear and damage caused by use

IMPORTANT!

Time for reprocessing

Preparation for cleaning, pre-cleaning and automated cleaning and disinfection has to be done immediately after the application, however not later than 1 hour after the application

Cleaning and disinfection

WARNING!

The complete cleaning and disinfection process of the Handpiece A9000 is performed **together with the Connecting cable A9000**, as described below, **attached to the Handpiece A9000**:

To avoid any penetration of moisture during the complete cleaning process into the electrical connectors of the Handpiece A9000 and the Connecting cable A9000, **please note**:

- The connection plug of the Handpiece A9000 must be covered with the free immersion cap on the Connecting cable A9000.
- Both electrical terminals of the Connecting cable A9000 must be covered by the immersion caps on the ends of the Connecting cable A9000

NON-COMPLIANCE WITH THESE INSTRUCTIONS CAN LEAD TO OPERATING FAULTS!

ATTENTION!

- Do not twist the cable connector when inserting or removing it. It is a **push-pull connector** and can be damaged by twisting it
- To avoid damaging the cable, **only touch the grooved part of the cable plug when connecting and disconnecting the cable**
- Never pull on the cable itself

IMPORTANT!

Cancellation of the guarantee

- Carrying out a process different from the one indicated in this manual will invalidate the guarantee.
- Failure to comply with all or part of the process indicated in this manual will invalidate the guarantee

Manual pre-cleaning	Step 1	Rinse the external parts of the Handpiece A9000 and the Connecting cable A9000 for 5s under running tap water.
	Step 2	Brush the lumen of the suction channel three times with a suitable bottle brush and rinse with the aid of a water jet gun (1.5 to 2 bar pressure) with four water pulses of 5s each from the suction channel. Operate the valves three times during rinsing.
Automated cleaning	For automated cleaning, the suction ports of the Handpiece A9000 are connected to the flushing ports of the washer-disinfector. Parameters: <ul style="list-style-type: none"> - 2 minutes pre-cleaning with cold water from tap. - Draining - 5 minutes cleaning with tap water at 55°C and 0.5% (Neodisher Mediclean, Dr. Weigert GmbH) cleaning solution 	

	<ul style="list-style-type: none"> - Draining - 3 minutes rinsing with cold deionized water (rinsing / neutralization) - Draining
Disinfection	<ul style="list-style-type: none"> - 5 minutes thermal disinfection at 90°C with deionized water. - Draining
Remarks	<ul style="list-style-type: none"> - Temperature of cold tap water: 10°C – 25°C - Temperature of de-ionised cold water: 10°C – 25°C
Validated Workflow	The suitability of these medical devices for effective mechanical cleaning and disinfection was tested and proved by an independent and accredited test laboratory. A washer /disinfector G7836 CD (Co. Miele & Cie. KG, Gutersloh, Germany) and the detergent Neodisher Mediclean (0.5 %) (Dr. Weigert GmbH & Co. KG, Hamburg) were used. The above workflow was determined by the tests in the test laboratory under SMP-Bericht-Nr.: 04120-1 (Automated Thermal Disinfection) and 04120-2 (Automated Cleaning Process).

IMPORTANT!

When choosing the detergent, make sure that:

- It is suitable for cleaning medical devices with metal and plastic components.
- If thermal disinfection is not an option, a suitable disinfectant with proven suitability (e.g. approved by the German Association for Hygiene and Microbiology (DGHM) or the Association for Applied Hygiene (VAH), authorised by the FDA or provided with a CE mark) must be used. The disinfectant must be compatible with the detergent.
- All chemicals used during the process must be compatible with the present medical devices.

The concentrations specified by the manufacturers of the detergents and disinfectants have to be strictly observed.

Visual examination:

Visual examination of the state of cleanliness, if necessary, repeat the process until the product is visually clean. This assessment must be carried out each time reprocessing is carried out on the equipment.

- Examine equipment, including accessories, in terms of damage, loose or missing parts, hygiene and integrity.
- Verify the legibility and completeness of the labels necessary for their correct use.
- Check all connection cables for damage.

Packaging before sterilization

Once the cleaning and disinfection of the handpiece / connecting cable has been completed, the products must be checked and immediately packed in packaging suitable for sterilization (e.g. pouches or sterilization containers) in accordance with ISO 11607 and EN 868.

Sterilization

CAUTION!

The complete sterilization process is performed with the **Connecting cable A9000**, as described below, **attached with the Handpiece A9000**:

To avoid any penetration of moisture during the entire sterilization process into the electrical connectors of the Handpiece A9000 and the Connecting cable A9000, **please note**:

- The connection plug of the Handpiece A9000 must be covered with the free immersion cap on the Connecting cable A9000.
- Both electrical terminals of the Connecting cable A9000 must be covered by the immersion caps on the ends of the Connecting cable A9000.

NON-COMPLIANCE WITH THESE INSTRUCTIONS CAN LEAD TO OPERATING FAULTS!

ATTENTION!

- Do not twist the cable connector when inserting or removing it. It is a **push-pull connector** and can be damaged by twisting it.
- To avoid damaging the cable, **grasp only the grooved part of the cable connector when connecting and disconnecting the cable**.
- Never pull on the cable itself..

IMPORTANT!

Cancellation of the guarantee

- Carrying out a process different from the one indicated in this manual will invalidate the guarantee.
- Failure to comply with all or part of the process indicated in this manual will invalidate the guarantee.

WARNING!

Material damage due to incorrect sterilization method

Using a sterilization method other than that specified in these instructions for use (in particular plasma sterilization) may damage the product.

Sterilization process	Steam sterilization process (autoclave) before vacuum sterilization with wrapping in sterilization bags. Minimum parameters:	
	- Pre-vacuum steps	3
	- Sterilization temperature	132°C
	- Holding time (full cycle)	3 min
	- Drying time	10 min

Storage after reprocessing

After reprocessing and before the next use, store all products in the appropriate packaging in a clean, dry and dust-free environment protected from sunlight.

Reusability

The individual components can be reused as long as they function properly and the instruments can be properly secured, provided they are used with due care and are properly reprocessed, tested and packaged.

The (further) use of damaged or contaminated instruments is the sole responsibility of the user. The durability of the product depends to a large extent on careful handling during use and reprocessing, which is why the number of possible reprocessing cycles cannot be determined. The manufacturer accepts no liability if these instructions are not followed.

Troubleshooting

Problem	Possible cause	Possible solution
The Shaver Blade does not rotate	The Shaver Blade is not properly inserted in the Handpiece A9000	Remove the blade and reinsert it according to the instructions in the "Insertion and locking the Shaver Blades" section of this manual.
The A9000 Handpiece is not running	The Handpiece A9000 is not properly connected to the Control unit A9000	Remove the Connecting cable A9000 from the Control unit A9000 and Handpiece A9000 and reconnect it according to the instructions in this manual in the section "Connection of the Handpiece A9000"
	The Connecting cable A9000 has a malfunction	Send the Connecting cable A9000 to DEWIMED Medizintechnik GmbH for repair

Technical specifications

WARNING!

No modification of this equipment is allowed.

IMPORTANT!

Cancellation of the guarantee

Any modifications not authorized by the manufacturer will invalidate the guarantee.

Product group	O4 Shaver System A9000, SH-T001
Hazard class / standard (EC Directive 93/42/EEC)	Class IIa – Rule 9
EMDN number / description	Z1202110201 / MOTORIZED ARTHROSCOPIC SURGERY SYSTEMS
Article number	SH-T001.1.2
Article name	Handpiece A9000
Features	<ul style="list-style-type: none"> - Mounting coupling for REMA Shaver Blades Versions 0 and 1, REF 15-60XXX and 15-61XXX - Channel and connection for suction function - Rotation to the right or left - Oscillating (pendulum) motion (alternately left, right) - Speed regulation in steps of 100 rpm - Controllable via buttons on the handle, on the Control unit A9000 and via Foot switch A9000
Dimensions	L 250mm / W 55mm / H 55mm
Weight	450g
Running speed left/right	500 – 9000 min-1 +/- 2%
Oscillation speed	500 – 4000 min-1 +/- 2% t= 0.2, 0.4, 0.6 and 0.8 s
Supply voltage	24V DC
Power consumption	Max 75W
Degree of protection (IPXX)	IPX7 (with plug)
Service life	Up to 2 years, depending on frequency of use and proper handling
Operating instructions, including cleaning	UM-02 Instruction Manual Shaver System A9000 IFU-43
Sterilization	Steam sterilization: min. 132°C, max. 137°C Three minutes. (complete cycle), 10 minute drying time
Electric safety - standards	DIN EN 60601-1, DIN EN 60601-1-2
Materials, construction and testing	N/A, no surgical instruments, for further details see technical documentation
Biocompatibility	DIN EN ISO 10993-1
Packaging	Primary packaging: PE foam insert/carton board Outer packaging: bubble wrap, corrugated cardboard

Rücksendung

Returns will be accepted only if they are marked as "hygienically safe" or "not decontaminated" and have been securely packaged for shipping.

Attention!

Returned products must pass the complete reprocessing process before being returned for repair.

Liability / Warranty Claim / Repair and modification

DEWIMED Medizintechnik GmbH as manufacturer and seller of these products assumes no liability for immediate or ensuing damages resulting from improper use or handling, especially due to failure to use as directed or improper processing or due to combination with products of other manufacturers.

Never attempt to perform repairs yourself. Service and repair work may only be performed by persons qualified and trained accordingly. Defective handpieces A9000 may be sent to DEWIMED Medizintechnik GmbH for repair. Unauthorized modifications and repairs are strictly prohibited and will entail invalidation of the manufacturer's warranty and liability.

Disposal

The handpiece A9000 shall be disposed of in accordance with local regulations for recycling electrical and electronic equipment.

The handpiece A9000 complies with the requirements of Directive 2002/95/EC on the restriction of the use of certain dangerous substances in electrical and electronic equipment (RoHS).

RoHS/WEEE Environmental Legislation
(Only for selling in EU)

RoHS Compliance regarding Directive 2002/95/EC

Registered by EAR/Germany:
WEEE-Reg.-Nr. DE 78284835

ATTENTION!

Environmental protection

Improper disposal of the device or any of its components can have an impact on the environment

About this instructions of use

Throughout the period of use of medical devices, the Instructions for Use must be kept freely accessible for every user.

Explanation of the symbols

	Article number		Medical device		Quantity in packaging
	Serial number of the manufacturer		Manufacturer		Manufacturing date
	Follow instructions for use		Non-sterile		Waste electrical and electronic equipment
	Information on the degree of protection of mechanical and electrical enclosures against ingress, dust, accidental contact and water		Do not use if packaging is damaged		
	Attention		Prescriptive medical device (USA)		
	CE-Mark of the Notified Body mdc medical device certification GmbH, Stuttgart, Germany				

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Marque CE de l'organisme notifié mdc medical device certification GmbH, Stuttgart, Allemagne